



# **New Solutions to Enhance Testing Efficiency and Blood Safety**

**Jeffrey M. Linnen, Ph.D.**  
**Vice President, NAT Assay Development**  
**Grifols Diagnostic Solutions Inc.**  
**San Diego, California USA**

**IPFA/PEI 27<sup>th</sup> International Workshop on Surveillance and Screening of Blood-borne Pathogens – Virtual**  
**06 May 2021**



**GRIFOLS**

# Disclaimers


- Procleix Plasmodium assay and Procleix ArboPlex assay are under development and performance characteristics have not been determined; both assays are not available for commercial sale
- Procleix is a trademark of Grifols Worldwide Operations Limited
- Panther is a trademark of Hologic, Inc.

# Topics


- Grifols NAT Research & Development update
- New Assays under development for use on the Procleix Panther system:
  - Procleix Plasmodium assay: new assay for detection of 5 species of *Plasmodium*
  - Procleix ArboPlex assay: new multiplex assay for detection of West Nile, Usutu, chikungunya, dengue and Zika viruses



Centers for Disease Control and Prevention's Public Health Image Library

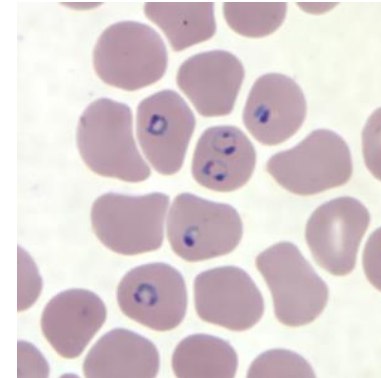

[www.cdc.gov/dpdx/index.html](http://www.cdc.gov/dpdx/index.html)

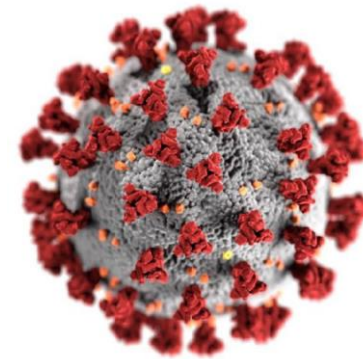

[www.cdc.gov](http://www.cdc.gov)



Scott Bauer. (USDA ARS)

# Grifols NAT Research & Development Update



- Procleix UltrioPlex E assay
  - Multiplex assay for detection of 5 viruses: HIV-1, HIV-2, HCV, HBV, and HEV
  - Launched in Japan for nation-wide screening: August 2020
  - CE mark: January 2021
- Procleix Babesia assay
  - Detection of 4 *Babesia* species: *B. microti*, *B. divergens*, *B. duncani*, and *B. venatorum*
  - US FDA approval: January 2019; CE mark: January 2021
- Procleix Panther featuring ART (Automation Ready Technology)
  - Significant hardware & software improvements to decrease “hands-on” time and total labor hours
  - Simplified user management through connection of multiple instruments to track system and control from central dashboard
  - CE mark: October 2019; US FDA approval: April 2020
- Procleix SARS-CoV-2 assay
  - Same technology as blood screening assays (magnetic-based target capture, TMA, and chemiluminescent detection) with modifications for respiratory specimen testing
  - Available commercially in selected EU markets; organ-tissue donor screening in US (under US FDA Emergency Use Authorization)
  - Research studies at Vitalant Research Institute & American Red Cross
  - CE mark: May 2020

Procleix Panther System 




**GRIFOLS**

# Procleix Plasmodium Assay (Under Development)

# Procleix Plasmodium Assay on Panther System

NAT under development for detection of *Plasmodium* ribosomal RNA

- Requires whole blood sample lysis step performed manually or on automated pipettor (Procleix Xpress System)
  - Whole blood added to Parasite Transport Medium (PTM) to disrupt red blood cells, parasites, and stabilize *Plasmodium* 18s rRNA
  - Sample preparation identical to US FDA licensed and CE-marked Procleix Babesia assay
  - Testing of individual or pooled lysates (16-lysate samples)
- Assay designed to detect five *Plasmodium* species with equivalent sensitivity:
  - *P. falciparum*, *P. knowlesi*, *P. malariae*, *P. ovale*, *P. vivax*
- RUO version of assay currently used in research study to evaluate donors deferred for malaria risk

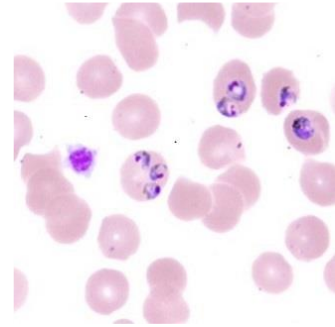

# Research Study: Malaria Risk in Deferred Donors

## Two Phases using RUO version of Procleix Plasmodium assay

### Phase 1

#### Preliminary Clinical Specificity

- 10,000 – 20,000 unique, unlinked US donations
- Testing at American Red Cross (ARC, Gaithersburg, MD)
- Initially reactive results:
  - Re-tested in duplicate using the same lysate
  - New lysate tested in triplicate
  - Further confirmatory testing with alternative NAT, antigen, and antibody detection
- Confirmed positive samples to be evaluated in lysate pools of 8 and 16

### Phase 2

#### Screening Deferred Donors

- 5,000 deferred donors collected in US and Canada
- Testing at American Red Cross (ARC, Gaithersburg, MD USA)
- 3 individual lysates each tested in singlet to increase sensitivity
  - Estimated to allow 97% probability of detecting ~1 parasite/mL, based on a 95% limit of detection of ~3 parasites/mL
- Initially reactive results: re-tested same as Phase 1
- Confirmatory testing: same as Phase 1
- Confirmed positive samples to be evaluated in lysate pools of 8 and 16


# Preliminary Clinical Specificity of Procleix Plasmodium Assay

## Phase 1 Results

- 10,751 fresh whole blood donations collected by ARC: June 21, 2019 to January 16, 2020

# Specimens Tested	# Initial Reactive	# Nonreactive	# Confirmed Positive	% Specificity (95% CI)
10,751	0	10,751	0	100 (99.966 –100)

#: number; CI: Confidence Interval

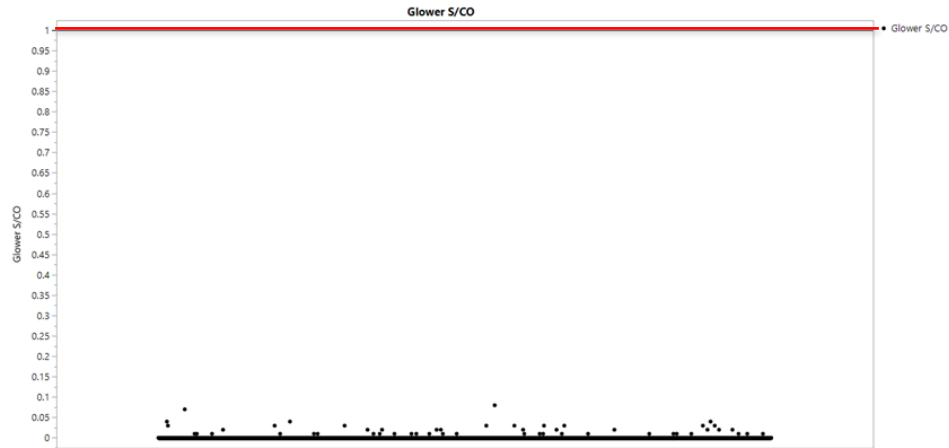

**100% specificity observed in normal donors collected in the US**




# Plasmodium RNA in US Donors Deferred for Malaria Risk

## Phase 2 (interim results)

- 315 deferred US donors tested with Procleix Plasmodium RUO Assay (3 individual lysates tested)
  - ~15% born in the US / ~85% born outside of the US (India ~33%, Brazil ~16%, Mexico ~8%, Philippines and China ~4%, Columbia ~3%)
  - ~80% never been in Africa / ~20% lived or traveled to Africa

# Donations Tested	# Initial Reactive	# Nonreactive	# Confirmed Positive
315	0	945	0

#: number; CI: Confidence Interval

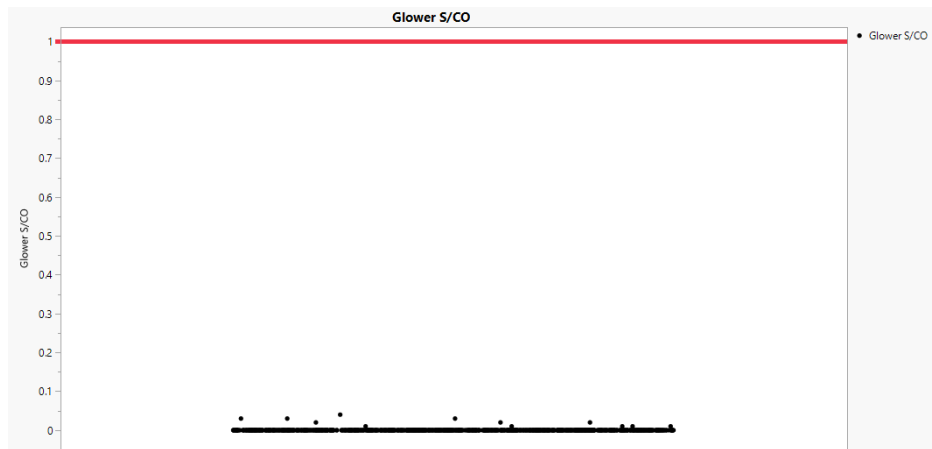

**No reactive donors were identified among deferred donors tested thus far**

# Procleix Plasmodium Assay

## Preliminary analytical sensitivity (RNA copies/mL)

Limit of Detection (LoD) by probit analysis using *in vitro* synthesized transcripts for 5 Plasmodium species in copies/mL

<i>In vitro</i> transcript, N=24	50% LoD in Copies/mL (95% Fiducial Limits)	95% LoD in Copies/mL (95% Fiducial Limits)
<i>P. falciparum</i>	1.9 (1.0 - 2.7)	7.0 (4.6 – 17.4)
<i>P. knowlesi</i>	4.9 (3.5 - 6.4)	12.5 (9.2 – 21.0)
<i>P. malariae</i>	2.3 (1.2 – 3.3)	11.6 (7.3 – 29.7)
<i>P. ovale</i>	6.7 (4.9 – 8.3)	12.3 (10.1 – 18.5)
<i>P. vivax</i>	4.5 (3.0 – 6.2)	16.3 (11.1 – 32.5)

- Similar LoD values for 5 species tested
- 95% LoD ranged from 7.0 to 16.3 copies/mL (similar to other Procleix assays)

# Procleix Plasmodium Assay

## Preliminary analytical sensitivity (parasitized RBCs/mL)

- Cultured *P. falciparum* infected erythrocytes and naturally infected specimens for 4 species
- Infected erythrocytes quantified by Fluorescence-Activated Cell Sorting (FACS)
- Naturally infected human whole blood quantified with research real-time Plasmodium TMA assay

Sample type, N=16	50% LoD in pRBC/mL (95% Fiducial Limits)	95% LoD in pRBC/mL (95% Fiducial Limits)
<i>P. falciparum</i> -infected erythrocytes	0.6 (0.3 - 0.9)	2.6 (1.7 – 6.3)
<i>P. falciparum</i> *	0.4 (0.0 – 0.7)	3.2 (1.9 – 11.1)
<i>P. malariae</i> *	0.7 (0.3 – 1.1)	3.3 (2.2 – 6.9)
<i>P. ovale</i> *	1.5 (1.0 – 2.0)	5.0 (3.6 – 9.3)
<i>P. vivax</i> *	0.9 (0.5 – 1.2)	2.8 (2.0 – 5.8)

Cultured *P. falciparum* infected erythrocytes obtained from New York Blood Center; \* Naturally Infected Human Whole Blood obtained from Wadsworth Center, NYSDOH

- Similar LoD values for 4 species tested
- 95% LoD ranged from 2.6 to 5.0 pRBC/mL (similar to Procleix Babesia Assay)

# Procleix Plasmodium Assay

## Preliminary analytical sensitivity (IU/mL): 1st WHO International Standard (04-176)

- Freeze-dried whole blood preparation collected from a patient by exchange transfusion
- Resuspended in 0.5 mL of nuclease-free water for 20 min; concentration  $1 \times 10^9$  IU/mL
- Half-log dilutions from 10,000 IU/mL to 3.16 IU/mL

1 <sup>st</sup> WHO IS (04-176), IU/mL	% Reactivity, n = 24
10,000	100
3,160	100
1,000	100
316	100
100	100
31.6	100
10	79
3.16	33
0	0

**Note:** the Plasmodium 1<sup>st</sup> WHO IS was developed for nucleic acid tests targeting **DNA**. Procleix Plasmodium assay targets **RNA**; handling of the material is critical


1 <sup>st</sup> WHO IS (04-176)	50% LoD in IU/mL (95% Fiducial Limits)	95% LoD in IU/mL (95% Fiducial Limits)
<i>P. falciparum</i>	5.0 (2.7 – 6.9)	16.6 (11.2 – 44.3)

# Procleix ArboPlex Assay (Under Development)

# Summary of Design Goals for the ArboPlex Assay

Assay under development for detecting 5 arboviruses

- Detect 5 viruses and internal control (IC) in one reaction:
  - Dengue viruses (DENV): types 1-4
  - Chikungunya virus (CHIKV): West African, East/Central/South African, and Asian genotypes
  - Zika virus (ZIKV): African and Asian lineages
  - West Nile virus (WNV): lineages 1 and 2 and Usutu virus (USUV)
- Use existing technology: magnetic-based target capture, TMA, and chemiluminescent detection
  - Compatible with existing Panther instruments (or Panther ART) at current testing sites; no new hardware or major system SW changes needed
- No increase in staffing needs—no increase in turn-around time
- Detection of WNV/USUV from initial reactive result (no discrimination)
- Maintain or improve performance compared to the Procleix Dengue, Zika and WNV assays

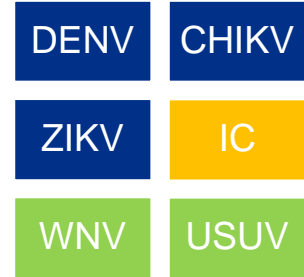

**GRIFOLS**

# Procleix ArboPlex Assay

## Preliminary analytical sensitivity: WNV

- Lineage 1: virus in processed plasma (defibrinated, delipidated citrate sodium plasma)
- Lineage 2: *in vitro* synthesized transcript

WNV Lineage 1 TCID <sub>50</sub> /mL	ArboPlex Assay		Procleix WNV Assay	
	# Reactive / # Tested	% Reactivity	# Reactive / # Tested	% Reactivity
0.692	20/20	100%	20/20	100%
0.347	20/20	100%	20/20	100%
0.115	19/20	95%	20/20	100%
0.035	13/20	65%	14/20	70%
0.011	5/20	25%	10/20	50%
0	0/20	0%	0/20	0%

WNV Lineage 2 Copies/mL**	ArboPlex Assay		Procleix WNV Assay	
	# Reactive / # Tested	% Reactivity	# Reactive / # Tested	% Reactivity
60	20/20	100%	20/20	100%
30	20/20	100%	20/20	100%
10	19/20	95%	16/20	80%
3	12/20	60%	5/20	25%
1	1/20	5%	2/20	10%
0	0/20	0%	0/20	0%

\*\* RNA *in vitro* transcript (IVT)

Assay	Lineage (strain)	Detection Probabilities, TCID <sub>50</sub> /mL or Copies/mL	
		50% (95% Fiducial Limit)	95% (95% Fiducial Limit)
ArboPlex	Lineage 1 (NY2001-6263)	0.025 (0.013-0.037)	0.110 (0.069-0.269)
Procleix WNV Assay		0.014 (0.004-0.022)	0.079 (0.047-0.347)
ArboPlex	Lineage 2 (Hungary 2004)	3.13 (2.03 – 4.28)	8.86 (6.30 – 15.86)
Procleix WNV Assay		5.27 (3.49 – 7.35)	16.23 (10.94 – 35.15)

# Procleix ArboPlex Assay

Preliminary Analytical Sensitivity: WNV lineages 1 and 2 and Usutu virus

Lineage (Strain)	TCID <sub>50</sub> /mL or c/mL	% Reactive	
		ArboPlex Assay	Procleix WNV assay
WNV L1 (NY2001-6263)	0.347 TCID <sub>50</sub> /mL	100%	100%
	0.115 TCID <sub>50</sub> /mL	90%	100%
WNV L1* (NY99)	30 c/mL	90%	100%
	10 c/mL	90%	90%
WNV L2* (2014/Hungary)	30 c/mL	90%	100%
	10 c/mL	40%	10%
WNV L2* (Italy/Montova/40.1)	30 c/mL	90%	50%
	10 c/mL	80%	10%
WNV L2 (1986)	0.036 TCID <sub>50</sub> /mL	100%	100%
	0.012 TCID <sub>50</sub> /mL	80%	100%
WNV L2 (B-956 Uganda)	0.479 TCID <sub>50</sub> /mL	100%	100%
	0.162 TCID <sub>50</sub> /mL	100%	80%
USUV Europe 3* (1477)	100 c/mL	100%	NA
	30 c/mL	90%	
USUV African 3* (491)	100 c/mL	100%	
	30 c/mL	70%	

\* RNA *in vitro* transcript (IVT); NA = Not applicable due to USUV panel levels being below the limit of detection with Procleix WNV assay



# Procleix ArboPlex Assay

Preliminary analytical sensitivity: WHO International Standard ZIKV (11468/16)

ZIKV PEI Code 11468/16, IU/mL*	ArboPlex Assay		Procleix Zika Virus Assay	
	# Reactive / # Tested	% Reactivity	# Reactive / # Tested	% Reactivity
30	20/20	100%	NT	NT
10	20/20	100%	20/20	100%
3	19/20	95%	20/20	100%
1	14/20	70%	16/20	80%
0.3	4/20	20%	8/20	40%
0	0/20	0%	0/20	0%

\*Diluted in processed plasma: defibrinated, delipidated citrate sodium plasma; NT = Not tested

Assay	Detection Probabilities, IU/mL	
	50% (95% Fiducial Limit)	95% (95% Fiducial Limit)
ArboPlex	0.70 (0.38 – 1.01)	2.71 (1.80 – 5.96)
Procleix Zika Virus Assay	0.42 (0.17 – 0.63)	1.69 (1.09 – 5.39)
Procleix Zika Virus Assay Package Insert*	0.64 (0.54 – 0.76)	2.90 (2.22 – 4.18)

\* tested in negative plasma

# Procleix ArboPlex Assay

## Preliminary analytical sensitivity: Dengue virus types 1 & 4

- DENV-1 and DENV-4 *in vitro* synthesized transcripts tested

DENV-1 Copies/mL	ArboPlex Assay		Procleix Dengue Virus Assay	
	# Reactive / # Tested	% Reactivity	# Reactive / # Tested	% Reactivity
100	20/20	100%	20/20	100%
30	20/20	100%	20/20	100%
10	16/20	80%	15/20	75%
3	9/20	45%	9/20	45%
1	7/20	35%	3/20	15%
0	0/20	0%	0/20	0%

DENV-4 Copies/mL	ArboPlex Assay		Procleix Dengue Virus Assay	
	# Reactive / # Tested	% Reactivity	# Reactive / # Tested	% Reactivity
100	20/20	100%	20/20	100%
30	19/20	95%	19/20	95%
10	17/20	85%	10/20	50%
3	8/20	40%	6/20	30%
1	4/20	20%	0/20	0%
0	0/20	0%	0/20	0%

Assay	DENV Type	Detection Probabilities, Copies/mL	
		50% (95% Fiducial Limit)	95% (95% Fiducial Limit)
ArboPlex	DENV-1	2.80 (1.28 – 4.46)	20.28 (11.69 – 62.84)
Procleix Dengue Virus Assay		4.16 (2.41 – 6.11)	19.12 (12.06 – 45.83)
ArboPlex	DENV-4	3.81 (2.00 – 5.88)	23.91 (14.54 – 57.73)
Procleix Dengue Virus Assay		8.66 (5.65 – 12.16)	30.95 (20.76 – 62.01)

# Procleix ArboPlex Assay

Preliminary Analytical Sensitivity: Dengue virus types 1-4 (*in vitro* synthesized transcripts)

DENV Type (Strain)	Copy/mL	% Reactive, n = 10	
		ArboPlex Assay	Procleix Dengue Assay
Type 1 (Hawaii)	100	100%	100%
	30	100%	90%
	10	90%	90%
Type 2 (New Guinea C)	100	100%	100%
	30	100%	100%
	10	70%	70%
Type 3 (80-2)	100	100%	100%
	30	100%	100%
	10	90%	80%
Type 4 (H241)	100	100%	100%
	30	90%	100%
	10	80%	80%

# Procleix ArboPlex Assay

Preliminary Analytical Sensitivity: WHO International Standard Chikungunya Virus (11785/16)

CHIKV PEI Code 11785/16, IU/mL*	ArboPlex Assay		Real-Time Triplex Research Assay (RUO)	
	# Reactive / # Tested	% Reactivity	# Reactive / # Tested	% Reactivity
30	20/20	100%	20/20	100%
10	19/20	95%	18/20	90%
3	15/20	75%	9/20	45%
1	6/20	30%	5/20	25%
0.3	4/20	20%	0/20	0%
0	0/20	0%	0/20	0%

\*Prepared in processed plasma (defibrinated, delipidated citrate sodium plasma)

Assay	Detection Probabilities, IU/mL	
	50% (95% Fiducial Limit)	95% (95% Fiducial Limit)
ArboPlex Assay	1.55 (0.86 – 2.37)	9.27 (5.60 – 22.83)
Real-Time Triplex Research Assay (RUO)	3.28 (2.13 – 4.66)	12.32 (8.13 – 25.29)

# Procleix ArboPlex Assay

Preliminary Analytical Sensitivity: CHIKV East/Central/South African, West African, Indian Ocean and Asian lineages

CHIKV lineage* (Strain)	Copies/mL	% Reactive, n =10
East/Central/South African (S27)	100	100%
	30	100%
	10	90%
West African (SH2830)	100	100%
	30	100%
	10	60%
Indian Ocean (BNI-CHIKV 899)	100	100%
	30	100%
	10	100%
Asian (99659)	100	100%
	30	100%
	10	80%

\*in vitro synthesized transcripts

# Summary

- Grifols continues to develop multiplex assays to consolidate testing into a single reaction
  - Procleix UltrioPlex E assay (HIV-1, HIV-2, HCV, HBV, and HEV)
  - Procleix ArboPlex assay (WNV, USUV, DENV, ZIKV, and CHIKV)
- Panther ART will allow advances in automation
  - Increase in efficiency possible by decreasing hands-on time and centralized control of multiple instruments
- Procleix Plasmodium assay
  - Demonstrated high specificity and sensitivity (both in RNA copies/mL and parasitized red cells)
  - Sensitive detection of ribosomal RNA *could* change current testing and deferral strategies (possible elimination of antigen or antibody detection assays)
- Procleix ArboPlex assay
  - Preliminary results using a wide range of genetic variants showed similar or improved analytical sensitivity compared to current monoplex assays
  - Multiplex testing will help address the unpredictable nature of arboviral outbreaks

# Acknowledgements

## American Red Cross (ARC)

- Susan Stramer
- Laura Tonnetti
- Bryan Spencer
- Jamel Groves
- David Krysztof

## Héma-Québec

- Gilles Delage
- Yves Grégoire

## New York Blood Center

- Debra Kessler

## Grifols Diagnostic Solutions Inc.

- **R&D:** Vanessa Brès, Kristin Livezey, Ed Ong, Mimi Broderson, Jasmine Cooper, Alan Dickinson, Adrian Gurrola, Anthony James, Alanna Janssen, Sunghae Joo, Novelle Kimmich, James Knight, Daren Le, Kimberly Mejia, Lansha Peng, Sarah Arroyo, Deanna Self, Tim Shin, Zeqing Xu, Jerry Yuan
- **Program Management:** Marla Sanders
- **Manufacturing Sciences:** Stacy Fuller, Jennifer Ly, Marijane Jimenez, Thea Landau, Leonora Samia, Carmen Soikowski
- **Regulatory Affairs:** Amanda Doe
- **Instrument Support:** Daniel Robinson and Manuel Duarte

# Thank You!

